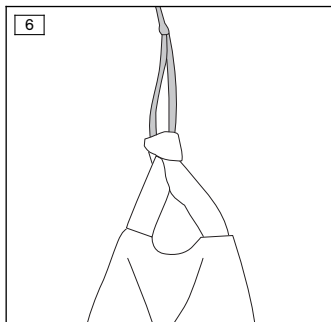
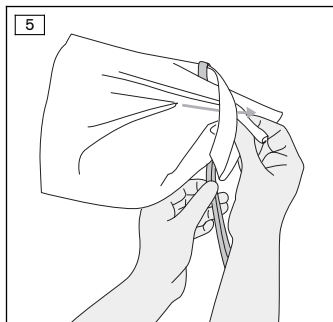
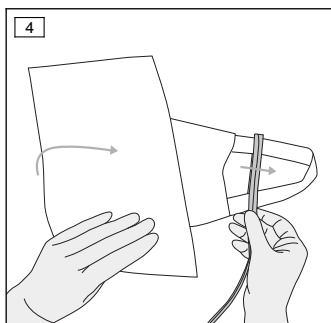
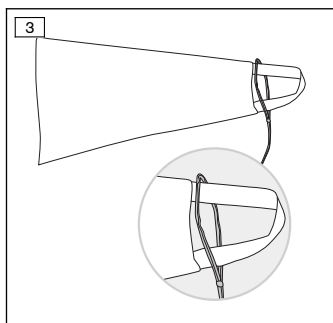
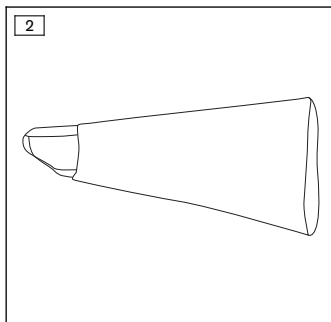
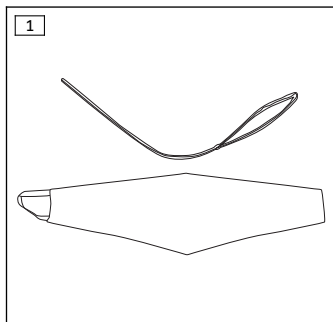
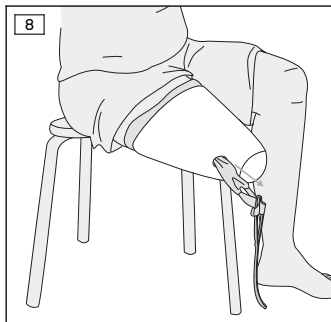
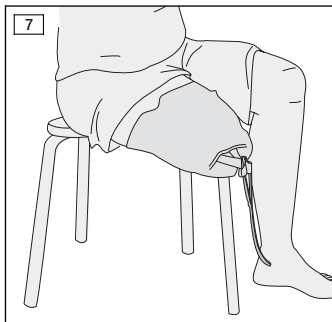
ottobock.

OC1560 EasyFit

DE Gebrauchsanweisung	4
EN Instructions for use	7
FR Instructions d'utilisation	10
IT Istruzioni per l'uso	13
ES Instrucciones de uso	16
PT Manual de utilização	19
NL Gebruiksaanwijzing	22
SV Bruksanvisning	25
DA Brugsanvisning	28
NO Bruksanvisning	31
FI Käyttöohje	33
PL Instrukcja użytkowania	36
HU Használati utasítás	39
CS Návod k použití	42
RO Instrucțiuni de utilizare	45
HR Upute za uporabu	48
SL Navodila za uporabo	51
SK Návod na používanie	54
BG Инструкции за употреба	57
TR Kullanma talimatı	60
EL Οδηγίες χρήσης	63
RU Руководство по применению	66
JA 取扱説明書	69
ZH 使用说明书	72
KO 사용 설명서	75

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Last update: 2018-05-04

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The OC1560 EasyFit donning sheath facilitates donning of transfemoral prostheses featuring a valve opening.

It is made of a lightweight material (silicone-coated nylon) with very good sliding qualities and reduces the pain prosthesis wearers may experience when donning their prosthesis.

2 Intended use

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

The product can be used for prostheses with a valve opening.

3 Safety

3.1 Explanation of warning symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General safety instructions

 **WARNING**

Product within reach of children

Danger of suffocation due to lack of oxygen supply

- ▶ Store the product away from the reach of children.

⚠ CAUTION

Using the product without following the instructions for use

Deteriorating health condition and damage to the product because of failure to follow the safety information

- ▶ Observe the safety information in these instructions for use.

⚠ CAUTION

Reuse on other users

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Use the product on only one user.

NOTICE

Mechanical overload

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

4 Handling

4.1 Application

NOTICE

Use of powder (e.g. baby powder, talcum)

Loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Avoid contact with powders.

- 1) Fully spread out the donning sheath and the accompanying pull-through aid (see fig. 1).
- 2) Turn one half of the donning sheath into the other half, so that the strap is facing out (see fig. 2).
- 3) Pull the strap on the donning sheath through the loop on the pull-through aid (see fig. 3).
- 4) Pull the wide end of the donning sheath through the gap between the strap and the loop (see fig. 4, see fig. 5).

- The pull-through aid and the strap of the donning sheath are now tied to each other (see fig. 6).
- 5) Pull the donning sheath with both hands over the transfemoral residual limb, avoiding any wrinkles (see fig. 7).
 - 6) Slide into the prosthetic socket and guide the pull-through aid through the valve opening on the prosthetic socket.
 - 7) Pull the donning sheath out through the valve opening on the prosthetic socket (see fig. 8).

INFORMATION

- ▶ Perform light pumping movements with the residual limb to aid donning the prosthetic socket.
- ▶ Pull on different parts of the material to ensure that the soft tissue is evenly distributed.

4.2 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Wash the product in **40 °C** warm water with standard mild detergent.
- 2) Allow the product to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal information

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document (Instructions for Use). The manufacturer will not assume liability for damage caused by disregard of this document.

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© Ottobock · 647G970-04-1805